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VORTEX

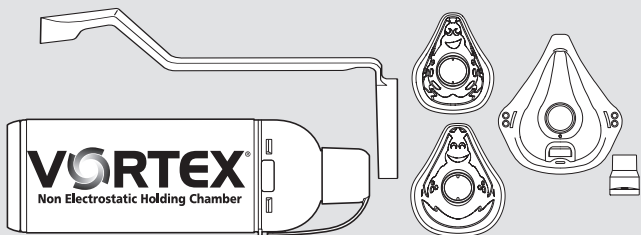
Model: VORTEX (Type 051)

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VORTEX Babymaske Käfer

VORTEX Kindermaske Frosch

PARI Erwachsenenmaske soft

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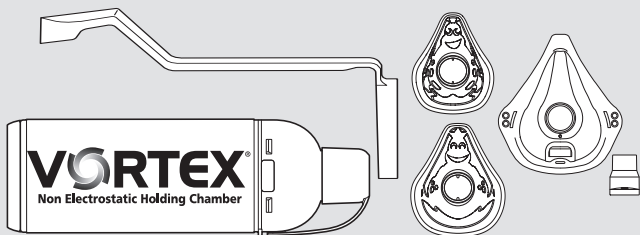
PARI adult mask soft

Read the instructions for use

Read these instructions for use carefully before using the product. Follow all instructions and safety instructions. Keep the instructions for use in a safe place.

Validity of instructions for use

PARI mask soft (Type 041)

VORTEX (Type 051)

VORTEX mask (Type 041)

Contact

E-mail: info@pari.de

Tel.: +49 (0)8151-279 220 (international)

+49 (0)8151-279 279 (German)

Copyright

©2022 PARI GmbH Spezialisten für effektive Inhalation
Moosstrasse 3, 82319 Starnberg, Germany. Technical and design changes and printing errors reserved. The same applies to illustrations.

Disclaimer

These instructions for use describe the components of PARI products and optional accessories. For this reason, these instructions for use also describe and illustrate features not present in your PARI product because they are, for instance, country-specific and/or optional. When using the systems, products and functions, the applicable country-specific regulations must be observed.

Trade marks

Registered trade marks of PARI GmbH Spezialisten für effektive Inhalation in Germany and/or other countries:

PARI®, VORTEX®

TABLE OF CONTENTS

| | |
|---|-----------|
| IMPORTANT INFORMATION | 5 |
| Intended purpose | 5 |
| Indication | 6 |
| Contraindications..... | 6 |
| Labelling | 7 |
| Safety and warning instructions | 8 |
| PRODUCT DESCRIPTION | 11 |
| Components | 11 |
| Overview and product designations | 11 |
| Description of function | 12 |
| Product combinations | 12 |
| Material information | 13 |
| Operating life | 14 |
| USE | 15 |
| Function check | 15 |
| Preparing for treatment | 16 |
| Performing treatment..... | 21 |
| REPROCESSING IN A HOME ENVIRONMENT | 25 |
| Reprocessing cycles | 25 |
| Processing limits | 25 |
| Preparation | 25 |
| Cleaning | 26 |
| Disinfecting | 27 |
| Inspecting | 30 |
| Drying | 30 |
| Storage..... | 30 |

| | |
|--|-----------|
| REPROCESSING IN PROFESSIONAL HEALTH INSTITUTIONS..... | 31 |
| Reprocessing cycles | 31 |
| Processing limits | 32 |
| VORTEX..... | 33 |
| PARI adult mask soft with mask connector | 38 |
| Visual inspection and storage | 44 |
| FURTHER INFORMATION | 45 |

1 IMPORTANT INFORMATION

1.1 Intended purpose

VORTEX

The VORTEX is a holding chamber. It is used in conjunction with medication sprays, the so called metered dose inhalers for treatment of the lower airways.

The VORTEX is designed for treatment of patients in all age groups (an appropriate mask is needed for treating babies and infants).

The VORTEX must only be used by individuals who understand the contents of the instructions for use and are able to operate the VORTEX safely.

Individuals in the following groups must be supervised by a person who is responsible for their safety:

- Babies, infants, and children
- Individuals with limited capabilities (e.g. physical, mental, sensory)

If the patient is not able to operate the VORTEX safely himself, the therapy must be carried out by the responsible person.

Only metered dose inhalers that have been approved for inhalation treatment must be used.

The VORTEX can be used in a home environment and in professional health institutions. When used in a home environment, the VORTEX is intended for single-patient use only (no patient change). In a professional environment, the device can be used with different patients as long as the corresponding hygiene reprocessing measures are observed.

The VORTEX is only designed for patients who are able to breathe by themselves and are conscious.

The frequency and duration of use is determined by professional medical staff¹ according to the individual needs of the patient.

1) Professional medical staff: Doctors, pharmacists, and physiotherapists.

Masks

The VORTEX masks are accessories for treatment using a PARI holding chamber. They enable inhalation of aerosol through the mouth and the nose.

The PARI adult mask soft is an accessory for nebuliser treatment. That enables inhalation of aerosol² through the nose and mouth.

The different mask sizes are suitable for treating patients in the following age groups:

- VORTEX baby mask Ladybug: Babies aged 0 to 2 years
- VORTEX child mask Frog: Children aged 2 to 4 years
- PARI adult mask soft: Adults

1.2 Indication

VORTEX

For treatment of diseases of the lower airways in combination with a metered dose inhaler.

Masks

For patients who cannot inhale using a mouthpiece, or if inhalation via mask is preferred.

Together with a holding chamber, the mask forms a system. The indication for this system is the same as the indication for the holding chamber used.

1.3 Contraindications

There are no contraindications known to PARI GmbH.

2) Aerosol: Small particles of solid, liquid, or mixed composition (fine "mist") suspended in gases or air.

1.4 Labelling

The following symbols can be found on the product and/or the packaging:

| | |
|---|---|
|  | Medical device |
|  | Legal manufacturer |
|  | Date of manufacture |
|  | This product conforms to the EU Medical Device Regulation 2017/745. |
|  | Consult instructions for use |
|  | Item no. |
|  | Production batch number, lot number |
|  | Unique Device Identifier (UDI) |
|  | Mask connector |
|  | VORTEX baby mask Ladybug |
|  | VORTEX child mask Frog |
|  | PARI adult mask soft |
|  | VORTEX |
|  | One-handed operation aid |

1.5 Safety and warning instructions

The present instructions for use contain important information, safety instructions, and precautionary measures. The user must follow these in order to guarantee safe operation of this PARI product.

This PARI product must be used only as described in these instructions for use.

The instructions for use of the metered dose inhaler used must also be followed.

Treatment of babies, infants, and anyone who requires assistance

For individuals who are not able to perform the therapy session without assistance or cannot appreciate the hazards, the risk of injury is greater. Such individuals include for example babies, children and people with limited capabilities. For these individuals, a person responsible for safety must supervise or carry out the application.

Labelling and classification of warning instructions

In these instructions for use, safety-critical warnings are categorised according to the following hazard levels:

DANGER

DANGER indicates a hazardous situation which will lead to very severe injuries or death if it is not avoided.

WARNING

WARNING indicates a hazardous situation which can lead to very severe injuries or death if it is not avoided.

CAUTION

CAUTION indicates a hazardous situation which can lead to mild or moderate injuries if it is not avoided.

NOTE

NOTE indicates a hazardous situation which can lead to material damage if it is not avoided.

General

Skin care products containing oils or fats can damage the soft plastic components. The patient should refrain from using skin care products of this kind while using the device.

If your health condition does not improve or it even worsens as a result of the treatment, seek professional medical advice.

Hazard due to small parts which can be swallowed

The product contains small parts. Small parts can block the airways and lead to a choking hazard. Keep all components out of the reach of babies and infants at all times.

Hygiene

Comply with the following hygiene instructions:

- Use only components that have been thoroughly cleaned and dried. Contamination and residual moisture encourage bacterial growth, which increases the risk of infection.
- Before every use and reprocessing cycle, wash your hands thoroughly.
- Make absolutely sure you also carry out reprocessing before using the device for the first time.
- Always use drinking water for reprocessing.
- Make sure all components are dried properly after each reprocessing step.
- Do not keep the components in a damp environment or together with damp objects.

Reporting serious incidents

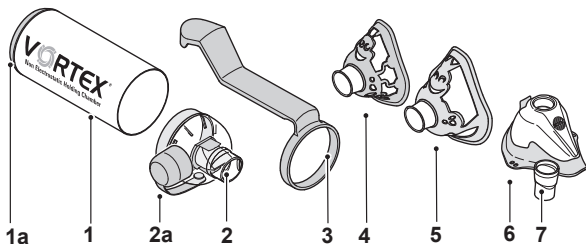
Report serious adverse incidents to the manufacturer and to the competent authority.

2 PRODUCT DESCRIPTION

2.1 Components

Please refer to the package for information on the supplied components.

2.2 Overview and product designations

| | |
|------|--|
| (1) | Inhalation chamber (antistatic) |
| (1a) | Connecting ring for metered dose inhaler (not removable) |
| (2) | Mouthpiece with inspiratory and expiratory valve |
| (2a) | Protective cap |
| (3) | One-handed operation aid |
| (4) | VORTEX baby mask Ladybug |
| (5) | VORTEX child mask Frog |
| (6) | PARI adult mask soft |
| (7) | Mask connector |

2.3 Description of function

The VORTEX helps the patient to inhale the medication from a metered dose inhaler evenly and completely.

The medication is sprayed into the inhalation chamber. There it is dispersed uniformly and can be inhaled completely by the patient in a few breaths. Consequently, it is not so important to coordinate the timing of releasing and inhaling the medication. It also prevents too much medication from being deposited in the mouth and throat. Side effects associated with premature deposition are avoided.

With the flexible connecting ring it can be used with most standard metered dose inhalers.

The VORTEX baby mask Ladybug, VORTEX child mask Frog and the PARI adult mask soft allow aerosols to be inhaled through the mouth and nose.

The mask connector functions as a connecting piece between the PARI adult mask soft and the VORTEX.

The one-handed operation aid is an optional component which makes using the VORTEX easier. With the one-handed operation aid, the VORTEX can be held and the metered dose inhaler actuated at the same time with one hand.

2.4 Product combinations

The VORTEX can be used with the following PARI masks:

- VORTEX baby mask Ladybug: Babies aged 0 to 2 years
- VORTEX child mask Frog: Children aged 2 to 4 years
- PARI adult mask soft: Adults who cannot inhale using a mouthpiece, or who prefer inhalation via mask

The mask type depends on the size and shape of the face. The ages shown are intended as guideline values.

2.5 Material information

The individual components are made from the following materials:

| | |
|--|---|
| Inhalation chamber | Aluminium |
| Mouthpiece | Polymethylpentene, polypropylene, thermoplastic elastomer |
| Inspiratory and expiratory valve in the mouthpiece | Silicone |
| Connecting ring for metered dose inhaler | Polypropylene, thermoplastic elastomer |
| Protective cap | Polypropylene |
| One-handed operation aid | Polypropylene |
| VORTEX baby mask Ladybug | Polypropylene, thermoplastic elastomer |
| VORTEX child mask Frog | Polypropylene, thermoplastic elastomer |
| PARI adult mask soft | Polypropylene, thermoplastic elastomer |
| Mask connector | Polypropylene |

2.6 Operating life

The individual product components have the following expected lifetimes:

| Product component | Service life |
|---|--|
| VORTEX and one-handed operation aid PARI adult mask soft, VORTEX baby mask Ladybug, VORTEX child mask Frog | In a home environment [see: Processing limits, page 25] |
| VORTEX PARI adult mask soft, VORTEX baby mask Ladybug, VORTEX child mask Frog | In a professional environment [see: Processing limits, page 32] |

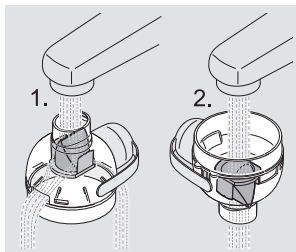
3 USE

All the steps described below must be carried out correctly.

3.1 Function check

After receiving the VORTEX, check that the path through the valve in the mouthpiece is unobstructed:

- Let the drinking water run out into the mouthpiece from the top.
The water should escape through the side openings in the mouthpiece.
- Let the drinking water run out through the bottom of the mouthpiece.
The water should run out of the lip attachment again.


- Dry the VORTEX completely before using it for the first time.
If the valve is obstructed, the VORTEX must not be used.

3.2 Preparing for treatment


Checking the valve position

Before every use, check that the valve is in the correct position inside the VORTEX mouthpiece:

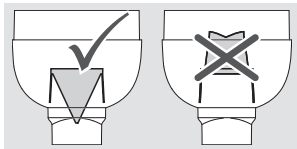- Check that the valve in the mouthpiece is in the position shown.

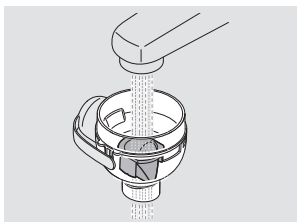- If necessary, use a stream of water to correct its position.

- Dry the VORTEX completely before using it.

Assembling the parts

WARNING

Risk of suffocation by breathing in small parts

The VORTEX inhalation chamber is not completely closed. As a result, there is a risk that small parts can get into it, which can be breathed in during inhalation.

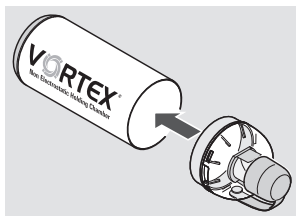
- Therefore, make sure there are no foreign bodies inside the VORTEX before every use.

CAUTION

Impaired treatment due to damaged or incorrectly assembled product components


Damaged components and an incorrectly assembled holding chamber may impair the function of the holding chamber and thus the treatment as well. Please also follow the instructions for assembly given below.

- Check all product components and the accessories before each use. Replace any broken, misshapen or seriously discoloured parts.
- Insert the mouthpiece in the inhalation chamber.


Use without one-handed operation aid

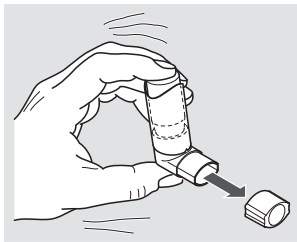- Remove the cap on the metered dose inhaler.
- Shake the metered dose inhaler vigorously before every use.

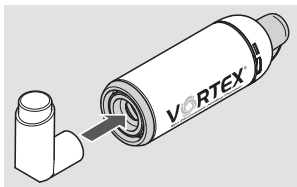
- Insert the metered dose inhaler in the connecting ring on the inhalation chamber.


Use with one-handed operation aid

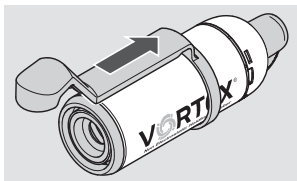
- Push the one-handed operation aid onto the inhalation chamber from the back.

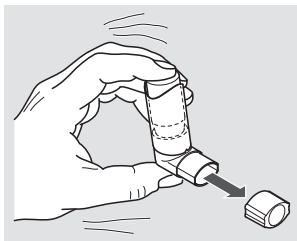- Remove the cap on the metered dose inhaler.
- Shake the metered dose inhaler vigorously before every use.

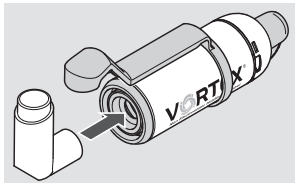
- Insert the metered dose inhaler in the connecting ring on the inhalation chamber.

- Position the one-handed operation aid so that its round end encloses the metered dose inhaler.

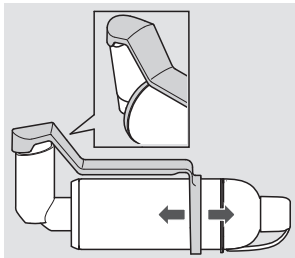
To do this, you may have to push the one-handed operation aid farther forwards or backwards.

i *Since the shape and size of metered dose inhalers has not been standardised, it is possible in some cases that the metered dose inhaler cannot be actuated with the one-handed operation aid. If this happens, the one-handed operation aid cannot be used. Please test before first application.*


Attach the mask (baby / child mask)

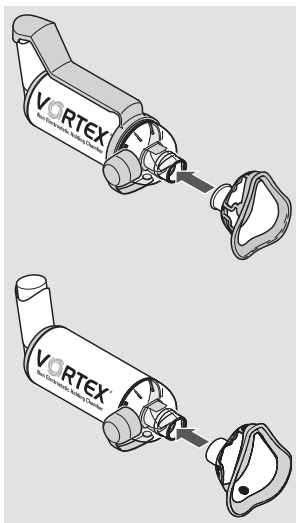- Take the protective cap off the mouthpiece of the VORTEX.
- Attach the mask to the VORTEX mouthpiece.


Attach the mask (adult mask)

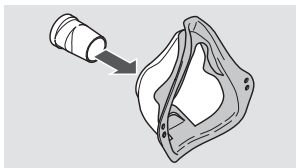- Take the protective cap off the mouthpiece of the VORTEX.
- Insert the mask connector in the opening in the PARI adult mask soft.

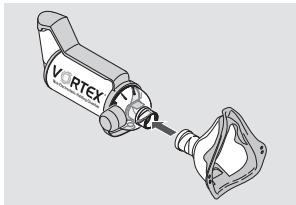- Attach the mask to the mouthpiece of the VORTEX with the mask connector.


⚠ CAUTION

Risk of medication underdose

If insufficient medication reaches the treatment site, the success of the treatment may be adversely affected.

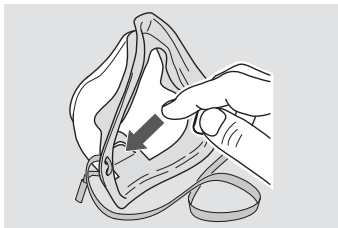- If the adult mask is being used, make sure that the expiratory valve shim is pressed out.

3.3 Performing treatment

All the safety instructions and warnings in these instructions for use must have been read and understood before any treatment is carried out.

⚠ CAUTION

Risk of underdosing with medication due to improperly assembled product components

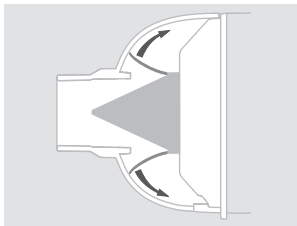
If insufficient medication reaches the treatment site, the success of the treatment may be adversely affected.

- Make sure before the treatment that all product components are firmly connected to each other so that no medication can escape.

Unless otherwise prescribed by your doctor, the following numbers of breaths are enough to inhale the medication from the VORTEX:


- Adults need one breath
- Children older than 18 months need two to four breaths
- Children younger than 18 months need four to six breaths

Tip for visual assessment of breathing function by people helping patients to perform the therapy: The lateral valves must open visibly when exhaling.

Inhaling with the mouthpiece

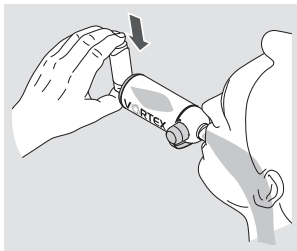- Take the protective cap off the mouthpiece of the VORTEX.
- Hold the mouthpiece between your teeth and enclose it with your lips.
- Begin breathing in slowly and deeply through the mouth. While breathing in, press the metered dose inhaler to release a puff of spray.
- Hold your breath briefly.
- Then breathe out slowly into the mouthpiece.

Inhaling with the baby and child mask

⚠ CAUTION

Impaired treatment due to escaping aerosol

If the mask does not form a seal on the face, aerosol may escape. This may result in medication underdosage.

- Make sure that the mask completely covers both corners of the mouth and the nose.
- Take note of possible side effects caused by escaping aerosol. These are described in the information for use of the respective medication.
- Place the VORTEX with mask on the face
- Observe your child's respiratory flow.

- Just before your child breathes in the next time, press the metered dose inhaler to release a puff.

Unless prescribed otherwise by the doctor:

With VORTEX

Baby mask Ladybug

Inhalation:

Four to six breaths in and out are to be taken through the mask.

With VORTEX

Inhalation through the child mask Frog:

Two to four breaths

in and out are to be taken through the mask.

- Allow your child to breathe back out through the mask.

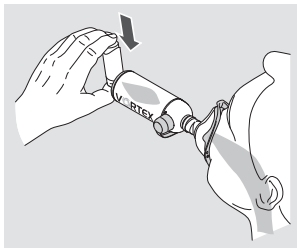
Inhaling with the adult mask

⚠ CAUTION

Impaired treatment due to escaping aerosol

If the mask does not form a seal on the face, aerosol may escape. This may result in medication underdosage.

- Make sure that the mask completely covers both corners of the mouth and the nose.
- Take note of possible side effects caused by escaping aerosol. These are described in the information for use of the respective medication.
- Place the VORTEX with mask on the face.
- Begin breathing in slowly and deeply through the mouth. While breathing in, press the metered dose inhaler to release a puff of spray.
- Hold your breath briefly.
- Then breathe out slowly through the mask.

Unless otherwise prescribed by your doctor, one breath is enough to inhale the medication from the VORTEX.

Use of the one-handed operation aid

The one-handed operation aid is used to make it easier to actuate the metered dose inhaler:

- Place your hand around the VORTEX so that your fingers are resting lightly on the one-handed operation aid.
- Press the one-handed operation aid down to release a puff of spray.
- Loosen your grip again.

4 REPROCESSING IN A HOME ENVIRONMENT

4.1 Reprocessing cycles

| | |
|--|--|
| VORTEX and one-handed operation aid | <ul style="list-style-type: none"> – Clean once a week. Clean once a day in the case of acute infection of the airways. Otherwise, whenever there is visible soiling. – Disinfect once per week. Clean once a day in the case of acute infection of the airways. |
| PARI adult mask soft, including mask connector | <ul style="list-style-type: none"> – Clean immediately after every use – Disinfect once per week |
| VORTEX baby mask Ladybug | |
| VORTEX child mask Frog | |


4.2 Processing limits

| | |
|---|-----------------------------------|
| VORTEX and one-handed operation aid, disinfection | 60 processing cycles, max. 1 year |
| PARI adult mask soft and mask connector, disinfection | 60 processing cycles, max. 1 year |
| VORTEX baby mask Ladybug, VORTEX child mask Frog | 60 processing cycles, max. 1 year |

4.3 Preparation

- Take the metered dose inhaler out of the connecting ring on the inhalation chamber and close it with the protective cap provided for this purpose.
- Dismantle the VORTEX into its individual components.

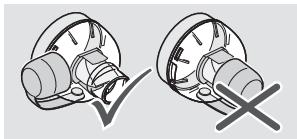Info: The protective cap must be open.

4.4 Cleaning

Precleaning

All individual parts must be pre-cleaned.

EQUIPMENT:

- Drinking water temperature of about 15 °C

PROCEDURE:

- Rinse all parts used for 2 minutes in running drinking water.

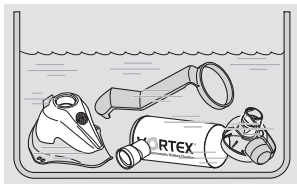
Manual cleaning

EQUIPMENT:

- Drinking water temperature of at least 40 °C
- Standard washing-up liquid³
- Receptacle with at least 3 l capacity

PROCEDURE:

- Add about 1 teaspoon of washing-up liquid to 3 l warm drinking water.
- Place all the parts in the washing-up water.
Application time: 5 minutes
- Occasionally move the parts back and forth.
- If there is visible soiling, use a medium-soft brush (such as a toothbrush) which is used exclusively for this purpose.

3) Validated with Palmolive®.

RINSING:

- Rinse all individual parts thoroughly under running drinking water at approx. 15 °C for 3 minutes.

DRYING:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean and absorbent surface, and allow them to dry completely.

In the dishwasher

The individual parts can be cleaned in a standard household dishwasher, provided it is connected to a mains water supply of drinking water quality.

To ensure safety when handling the cleaning agent used, follow the corresponding instructions for use, particularly the accompanying safety instructions.

PROCEDURE:

i *Do not clean the individual components together with very dirty dishes.*

- Place all components in the crockery basket so that no water can collect in them.
- Select a program running at 50 °C minimum.

DRYING:

Ensure that there is no residual moisture remaining in the components. If necessary:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean and absorbent surface, and allow them to dry completely.

4.5 Disinfecting

Disinfect all individual components after cleaning. Only components that have been cleaned can be disinfected effectively.

The validated disinfection procedures are described below.

In boiling water

EQUIPMENT:

- Clean saucepan
- Drinking water

PROCEDURE:

CAUTION

Risk of infection due to moisture

Moisture encourages the growth of bacteria.

- Remove all parts from the saucepan as soon as the disinfection process is complete, and allow them to dry.

When disinfecting in boiling water, the mouthpiece must be detached from the inhalation chamber. Use enough water to ensure as little contact as possible between the mouthpiece and the inhalation chamber. The inhalation chamber becomes very hot and might deform the mouthpiece.

- Place all the individual parts in water at a rolling boil for at least 5 minutes. **NOTE! Risk of damage to plastic parts!** Plastic will melt if it comes into contact with the hot base of the pan. Make sure there is enough water in the pan to prevent the individual parts from touching the pan base.

DRYING:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean and absorbent surface, and allow them to dry completely.

Using a standard thermal disinfecter for baby bottles (not a microwave oven)

EQUIPMENT:

- Thermal disinfecter with a runtime of at least 6 minutes

PROCEDURE:

CAUTION

Risk of infection due to inadequate disinfection

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection.

- Make sure that the disinfecter is clean and operating properly before every disinfection process.
- Allow the disinfection to continue until the disinfecter switches off automatically or the minimum disinfection time stated in the instructions for use of the disinfecter has elapsed. Do not switch the device off prematurely.

CAUTION

Risk of infection due to moisture

Moisture encourages the growth of bacteria.

- Remove all parts from the disinfecter as soon as the disinfection process is finished, and allow them to dry.

Regarding the performance of the disinfection, the duration of the disinfection procedure and the quantity of water required, follow the instructions for use of the disinfecter you are using.

DRYING:

- After the disinfection process is complete, place all parts on a dry, clean and absorbent surface and allow them to dry completely. Or else leave all individual parts in the closed thermal disinfecter for max. 24 hours until the next use.

4.6 Inspecting

Inspect all product components after each cleaning and disinfection. Replace any broken, deformed or seriously discoloured parts.

4.7 Drying

After each cleaning and disinfection, place all components on a dry, clean and absorbent surface, and let them dry completely. The inhalation chamber can be dried with a clean, lint-free cloth.

4.8 Storage

Store the VORTEX and the accessories, if applicable, as described below:

- When it has dried completely, push the mouthpiece into the inhalation chamber.
- Place the protective cap over the mouthpiece.
- Store the product and the accessories, if applicable, in a dry, dust-free place.

5 REPROCESSING IN PROFESSIONAL HEALTH INSTITUTIONS

5.1 Reprocessing cycles

Single patient use

| | |
|--|---|
| VORTEX | <ul style="list-style-type: none">- Clean once a week. Clean once a day in the case of acute infection of the airways. Otherwise, whenever there is visible soiling.- Disinfect once per week. Clean once a day in the case of acute infection of the airways. |
| PARI adult mask soft, including mask connector | <ul style="list-style-type: none">- Clean immediately after every use- Disinfect once per week |
| VORTEX baby mask Ladybug | |
| VORTEX child mask Frog | |

Before a change of patients

| | |
|--|---|
| VORTEX | <ul style="list-style-type: none"> – Cleaning – Disinfection – Sterilisation <p>Info: Sterilisation is not necessary if a dedicated mouthpiece is used exclusively by each patient.</p> |
| PARI adult mask soft, including mask connector | <ul style="list-style-type: none"> – Cleaning – Disinfection – Sterilisation |
| VORTEX baby mask Ladybug | Replace with a new mask. The mask cannot be sterilised. |
| VORTEX child mask Frog | |

5.2 Processing limits


| | |
|--|--|
| VORTEX, sterilisation | 30 processing cycles, max. 1 year |
| PARI adult mask soft and mask connector, Sterilisation | 100 processing cycles, max. 1 year |
| VORTEX baby mask Ladybug, VORTEX child mask Frog | Replace with a new mask. The masks cannot be sterilised. |

5.3 VORTEX

Preparation

- Take the metered dose inhaler out of the connecting ring on the inhalation chamber and close it with the protective cap provided for this purpose.
- Disconnect the mouthpiece from the inhalation chamber.

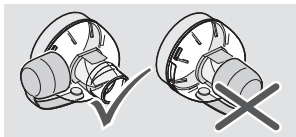Info: *The protective cap must be open.*

Precleaning

All individual parts must be pre-cleaned.

EQUIPMENT:

- Drinking water temperature of about 15 °C

PROCEDURE:

- Rinse all parts used for 2 minutes in running drinking water.

Cleaning and disinfection

Generally, a mechanical process (instrument dishwasher) should be used for cleaning and disinfection.

NOTE

Damage to the product caused by cleaning agents and disinfectants that are incompatible with the material

Not all cleaning agents and disinfectants are suitable for use on the materials and product components. Unsuitable agents can damage the product and render it unusable.

- Before cleaning and disinfecting, check that the cleaning agent or disinfectant used is compatible with the materials of the product components that are to be processed.

Mechanical cleaning and disinfection

EQUIPMENT

- Cleaning and disinfection device in conformance with DIN EN ISO 15883
- Cleaning agent
- Neutralising agent if necessary

PROCEDURE

- Arrange the components so that they can be cleaned most effectively.
The inhalation chamber must be held upright.
- Select a suitable cleaning programme and agent.

VALIDATION

The method has been validated in Europe using:

- RDG G7836 and CD G7736 MCU produced by Miele (programme Vario TD)
- Miele standard baskets (E142)
- Neodisher MediZym as a pH-neutral cleaning agent (0.5%, from Dr. Weigert)

Or:

- Neodisher Mediclean forte 0.5% (alkaline)
- For neutralising, use Neodisher Z (from Dr. Weigert)

DRYING

Even if the cleaner-disinfector is equipped with a drying function, make sure that no residual moisture is left in the components. If necessary, remove any residual moisture by shaking the parts and allow them to dry completely.

Chemical manual cleaning and disinfection

CAUTION

Risk of infection due to inadequate disinfection

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection.

- Observe the specified mixing proportions and the specified treatment time.
- Ensure that all components are completely submerged in the solution for the whole of the treatment time. There must not be any air pockets or bubbles.

Alternatively, cleaning and disinfection can also be carried out manually. The validated agents are listed together with all relevant information in the following table:

| | Korsolex® Endo Cleaner | Korsolex® Basic |
|--------------------|------------------------|-----------------------|
| Cleaning agent | × | – |
| Disinfectant | – | × |
| Contains aldehyde? | formaldehyde-free | × |
| Concentration | 0.5% | 4% |
| Duration | 5 min. | 30 min. |
| Temperature | 50°C | max. room temperature |

EQUIPMENT:

- Cleaning agent/Disinfectant
- Warm water
- Brush if applicable
- Basin

PROCEDURE:

If a disinfectant without a cleaning function is used, the following procedure must be performed twice: the first time with a cleaning agent, and then with the disinfectant.

To ensure safety when handling chemicals, follow the instructions for use of the agent used, particularly the accompanying safety instructions.

- Prepare the solution in the basin in accordance with the manufacturer's instructions. The quantity of solution required depends on the number of parts that are to be cleaned.
- Place all the individual parts in the prepared solution and leave them to soak.

Info: If the application period is exceeded significantly, the plastic parts may take on the smell of the disinfectant.

RINSING:

CAUTION

Risk of allergic reactions and irritation of the mucous membrane by disinfectants

Disinfectants can trigger allergic reactions or irritation of the mucous membrane on contact with the skin.

- Rinse the product thoroughly to ensure that no residues of the disinfectant remain on the PARI product.
- Rinse off all parts thoroughly in running drinking water.
Water temperature: approx. 15 °C, duration: 3 minutes

DRYING:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean, and absorbent surface, and allow them to dry completely.

Sterilisation

CAUTION

Risk of infection by residual germs

If any germs are left on the parts that are to be sterilised, they can proliferate despite sterilisation. As a result, there is a danger of infection.

- Clean, disinfect, and dry all parts thoroughly before sterilising.
- Use only validated procedures for cleaning and disinfection.

EQUIPMENT

Steam steriliser – preferably with fractionated pre-vacuum – in accordance with DIN EN 285 or DIN EN 13060 (Type B)

PROCEDURE

- Pack all of the disassembled parts in a sterile barrier system in accordance with DIN EN 11607 (e.g., foil-paper packaging).

Maximum sterilisation temperature and holding time:

132 °C / 134 °C, at least 3 min.

VALIDATION

The method has been validated in Europe using: Steam sterilisers Euro-Selectomat and Varioklav 400E

5.4 PARI adult mask soft with mask connector

Separated parts for processing

CAUTION

Risk of infection due to cross-contamination in the event of a change in patients

If a product is used for more than one patient, there is a risk that germs may be transmitted from one patient to the next.

- Clean, disinfect and sterilise all separated parts before every patient change.

The PARI adult mask soft and mask connector can be cleaned, disinfected, and sterilised according to the procedures described below.

Preparation

Dismantle the product into its individual parts.

Precleaning

All individual parts must be pre-cleaned.

EQUIPMENT:

- Drinking water temperature of about 15 °C

PROCEDURE:

- Rinse all parts used for 2 minutes in running drinking water.

Cleaning and disinfecting

Cleaning with disinfection

Cleaning and disinfection can be carried out in a single cycle using chemical reprocessing.

To ensure safety when handling chemicals, observe the instructions for use for the disinfectant.

Mechanical cleaning with disinfection:

EQUIPMENT:

The method has been validated in Europe using:

- Alkaline cleaning agent:
Dr. Weigert neodisher® MediClean forte (concentration: 0.5%)
- Deionised water
- Cleaning and disinfection device:
Steelco DS800 as per DIN EN ISO 15883-1 and 15883-2

Info: *If a different alkaline cleaning agent is used, it may also be necessary to use a neutralising agent. Follow the recommendations of the manufacturer of the chemical.*

PROCEDURE:

Program for cleaning and disinfecting according to manufacturer's instructions.

DRYING:

Ensure that there is no residual moisture remaining in the components.

- Shake the water out of all of the parts.
- Place all parts on a dry, clean, and absorbent surface, and allow them to dry completely.

**Chemical
cleaning with
disinfection:**

EQUIPMENT:

The method has been validated in Europe using:

- Aldehyde-free instrument disinfectant: Bode Bomix® plus (concentration: 2%)
Active agent basis: Quaternary ammonium compound
- Drinking water at about 15 °C
- Application time: 5 minutes

PROCEDURE:

- Clean and disinfect the individual parts in a single work step, using a solution prepared according to the manufacturer's instructions.
Info: *If the recommended treatment time is exceeded significantly, the plastic parts may take on the smell of the disinfectant.*

RINSING:

-  **CAUTION!** Disinfectant residues can cause allergic reactions or irritation of the mucous membrane. Rinse off all parts thoroughly in running drinking water at about 15 °C for 3 minutes.

DRYING:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean, and absorbent surface, and allow them to dry completely.

Chemical disinfection

To ensure safety when handling chemicals, observe the instructions for use for the disinfectant.

EQUIPMENT:

The method has been validated in Europe using:

- Aldehyde-containing disinfectant: Bode Korsorex® basic (concentration: 4%)
Active agent basis: Aldehyde donor, aldehyde
- Drinking water at about 15 °C
- Application time: 30 minutes

PROCEDURE:

CAUTION

Risk of infection due to growth of bacteria

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection.

- Observe the specified mixing proportions and the specified treatment time.
 - Ensure that all components are completely submerged in the solution for the whole of the treatment time. There must not be any air pockets or bubbles.
 - Disinfect the individual parts with a solution prepared according to the manufacturer's instructions.
- i** *If the recommended application period is exceeded significantly, the plastic parts may take on the smell of the medium used.*

RINSING:

CAUTION

Risk of allergic reactions and irritation of the mucous membrane by disinfectants

Disinfectants can trigger allergic reactions or irritation of the mucous membrane on contact with the skin.

- Rinse the product thoroughly to ensure that no residues of the disinfectant remain on the PARI product.
- Rinse off all parts thoroughly in running drinking water at about 15 °C for 3 minutes.

DRYING:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean and absorbent surface, and allow them to dry completely.

Sterilising

CAUTION

Risk of infection by residual germs

If there is dirt on the parts, germs capable of reproduction may remain despite the sterilisation process. As a result, there is a danger of infection.

- Clean, disinfect, and dry all parts thoroughly before sterilising.
- Use only validated procedures for cleaning and disinfection.

EQUIPMENT:

The method has been validated in Europe using:

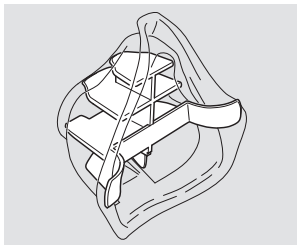
- Steam steriliser with fractionated pre-vacuum in accordance with DIN EN 285 or DIN EN 13060
- Sterile barrier system in accordance with DIN EN 11607-1
- Mask stabiliser for the PARI adult mask soft⁴
- Temperature: 132 °C/134 °C
- Holding time: min. 3 minutes

4) Mask stabilisers can be ordered from PARI GmbH separately.

PROCEDURE:

Information on sterilising a PARI adult mask soft:

Always use the appropriate mask stabiliser when sterilising, because otherwise it may lose its shape under the effects of high temperatures.

- Insert the mask stabiliser in the mask as shown in the illustration.
- Place all the disassembled parts in a sterile barrier system which complies with DIN EN 11607-1 (e.g. foil-paper packaging).
- Carry out the sterilisation in a steam steriliser in accordance with the manufacturer's instructions.

Sterilisation temperature and holding time:

132 °C/134 °C, at least 3 minutes.

5.5 Visual inspection and storage

Check all individual components. Replace any broken, deformed, or seriously discoloured parts.

- When it has dried completely, push the mouthpiece into the inhalation chamber.
- Place the protective cap over the mouthpiece.

Storage location:

- dry
- dust-free
- protected from sources of contamination

Optional: use sterile packaging

6 FURTHER INFORMATION

All product components may be disposed of with normal domestic waste. The country-specific disposal regulations must be observed.

All product components must be disposed of in accordance with the Ordinance on the European Waste Catalogue (EWC). Where applicable, country-specific disposal regulations must also be observed.

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4) Stabilizátory masek si můžete také objednat u společnosti PARI GmbH.
©2022 PARI GmbH Spezialisten für effektive Inhalation, 051D4013 A cs 2022-03-25