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PARI O-PEP

Model: PARI O-PEP (Type 018)

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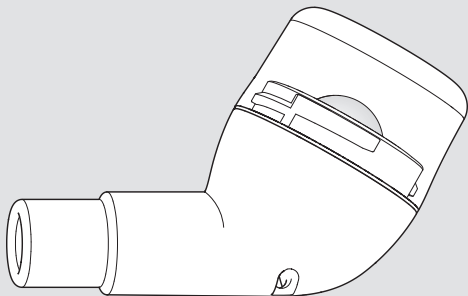
PARI O-PEP

Model: PARI O-PEP (Type 018)

Important: Read these instructions carefully before using the product for the first time. Follow all instructions and safety instructions!

Keep the instructions in a safe place.

Identification, validity, version

These instructions for use are valid for the PARI O-PEP (Type 018) in the following countries:

GB and countries outside the EU

Version of these instructions for use: Version A – 2020-03,
Approved version dated 2020-03-02

Information as of: 2020-03

The current version of the instructions for use can be downloaded from the internet as a PDF file:

www.pari.com (on the respective product page)

Formats available for visually impaired patients

The instructions for use available in PDF format on the internet can be enlarged for printing.

Conformity

The product conforms to the EU regulation 2017/745 on medical devices.

Trade marks

The following trade marks are registered trade marks of PARI GmbH Spezialisten für effektive Inhalation in Germany and/or other countries:

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No part of this documentation may be reproduced or processed, duplicated, translated or disseminated by electronic means without the prior, written consent of PARI GmbH.

All rights reserved. Technical and design changes and printing errors reserved. Principle shown in illustrations.

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Contact

For all product information and in the event of defects or questions about usage, please contact our Service Center:

Tel.: +49 (0)8151-279 220 (international)

+49 (0)8151-279 279 (German-speaking)

E-Mail: info@pari.de

TABLE OF CONTENTS

IMPORTANT INFORMATION	7
Intended purpose	7
Indication	7
Contraindications.....	7
Side effects.....	8
Safety instructions	8
PRODUCT DESCRIPTION	11
Components	11
Description of function	11
Material information	12
Operating life	12
APPLICATION	13
Assembling the PARI O-PEP	13
Perform treatment	14
REPROCESSING	16
Reprocessing cycles	16
Preparation	16
Cleaning and disinfecting	16
Care of the lanyard	20
Inspecting	20
Storage.....	21
MISCELLANEOUS	22
Disposal.....	22
Labelling	22
APPENDIX: Reprocessing in professional environments	23
Reprocessing cycles	23
Processing limits	23
Separated parts for processing	24
Preparation	24

Precleaning	24
Cleaning and disinfecting	25
Sterilising	29
Visual inspection and storage	30

1 IMPORTANT INFORMATION

1.1 Intended purpose

The PARI O-PEP (**O**scillating **P**ositive **E**xpiratory **P**ressure) is used for the mobilisation of secretion in cases of acute and chronic diseases of the lower airways.

The PARI O-PEP can be used:

- by children aged from 5 to 8 under the supervision and direction of a competent person
- by children aged 8 and older and adults after informed instruction

This PARI product can be used in a home environment and in professional health institutions. When used in a home environment, this PARI product is intended for single-patient use only (no patient change). In a professional environment, a multi-patient use is possible according to the reprocessing instructions.

This product must only be used by individuals who understand the contents of the instructions for use and are able to operate the product safely.

The frequency and duration of use will be determined by healthcare professionals¹ according to the individual needs of the patient.

1.2 Indication

Diseases of the lower airways which are associated with increased mucus build up.

1.3 Contraindications

The PARI O-PEP must not be used by individuals who are suffering from severe pneumothorax or coughing up substantial quantities of blood.

1) Professional medical staff: Doctors, pharmacists and physiotherapists.

1.4 Side effects

The following side effects can occur: Dizziness

If you notice any of these side effects, discontinue the treatment immediately. Contact your doctor or therapist to discuss how to proceed with treatment.

Inform your doctor or therapist if you notice any side effects that are not listed in these instructions for use.

1.5 Safety instructions

The present instructions for use contain important information, safety instructions and precautionary measures.

The user must follow these in order to guarantee safe operation of this PARI product.

This PARI product must only be used as described in these instructions for use.

General

The PARI O-PEP must not be used without first receiving instruction from a healthcare professional ¹.

The PARI O-PEP cannot be used with a mask. Masks have expiratory valves through which the exhaled air escapes, so it would not get into the PARI O-PEP. Respiratory therapy is therefore not possible with a mask.

If your health condition is not improved, or even worsens as a result of the treatment, seek advice from a healthcare professional¹.

Using the product

The PARI O-PEP is made from a material that is impact-resistant but not unbreakable. Therefore, protect it from impacts and take care to prevent it from falling.

Hazard due to small parts which can be swallowed

The product contains small parts. Small parts can block the airways and lead to a choking hazard. Keep all components of the product out of the reach of babies and infants at all times.

Hygiene

Observe the following hygiene instructions:

- Do not use product components unless they have been thoroughly cleaned and dried. Contamination and residual moisture encourage the growth of bacteria, which increases the risk of infection.
- Before every use and reprocessing cycle, wash your hands thoroughly.
- Make absolutely sure you also carry out reprocessing before using the device for the first time.
- Always use drinking water for reprocessing.
- Make sure all components are dried properly after each reprocessing step.
- Do not keep the product components in a damp environment or together with damp objects.

Reporting serious adverse events

Serious incidents which occur in connection with this PARI product must be reported promptly to the manufacturer and/or the distributor as well as to the competent authority. Any event which has led or might lead directly or indirectly to death or an unexpected serious worsening of the health condition of a person is to be considered serious.

Identifying and classifying warning instructions

Safety-critical warnings are categorised according to the following hazard levels in these instructions for use:

DANGER

DANGER indicates a hazardous situation which will lead to severe injuries or death if it is not avoided.

WARNING

WARNING indicates a hazardous situation which can lead to severe injuries or death if it is not avoided.

CAUTION

CAUTION indicates a hazardous situation which can lead to mild or moderate injuries if it is not avoided.

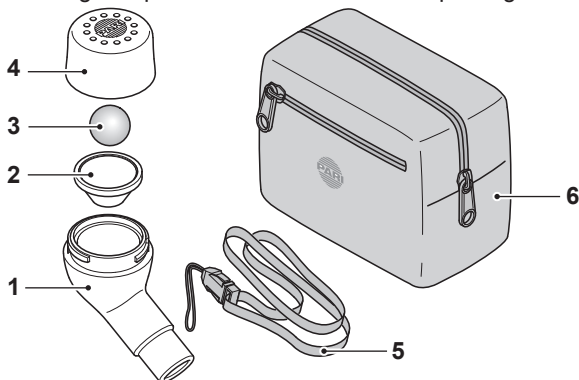
NOTE

NOTE indicates a situation which can lead to material damage if it is not avoided.

2 PRODUCT DESCRIPTION

2.1 Components

The following components are included in the package:

(1)	Lower part with mouthpiece
(2)	Funnel
(3)	Ball
(4)	Upper part
(5)	Lanyard
(6)	Case

2.2 Description of function

The patient exhales repeatedly into the device against a movable ball, causing a vibration that is transmitted to the lungs. This vibration loosens the mucus in the lower airways. During use, the airways are kept open and clearance of the loosened mucus is improved.

If the PARI O-PEP is held horizontally, the oscillation frequency during use is equal to about 15 Hz. This frequency can be varied by tilting the device upwards (higher frequency) or

downwards (lower frequency). This makes it possible to select the ideal frequency for each individual patient.

2.3 Material information

Lower part with mouthpiece, funnel, upper part	POM (polyoxymethylene)
Ball	Stainless steel
Lanyard	Polyester
Case	Polyester

2.4 Operating life

When used in home environments, the product has an expected lifetime of 300 disinfection cycles (max. 1 year). After that, the product must be replaced with a new one.

3 APPLICATION

People who assist others in carrying out the therapy must ensure that all of the steps described below are carried out correctly.

3.1 Assembling the PARI O-PEP

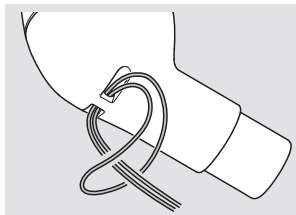
WARNING

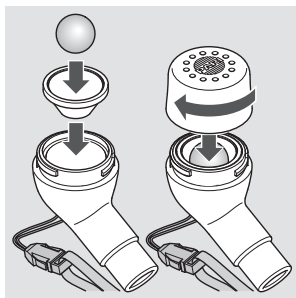
Risk of suffocation by breathing in small parts

The product is not completely closed. As a result, there is a danger that small parts can get inside it, and may be breathed in during inhalation.

- Therefore, it is imperative to make sure there are no foreign bodies inside the product before every use.
- Attach the lanyard to the lower part.

- Insert the funnel in the lower part.
- Place the ball in the funnel.
- Screw the upper part onto the lower part.

3.2 Perform treatment

All of the safety instructions in these instructions for use must have been read and understood before any treatment is carried out.

CAUTION

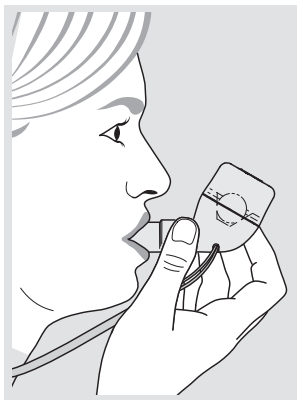
Risk of overexertion

Dizziness may occur due to overexertion during use. This may happen sooner with children because their lung capacity is smaller than that of adults.

- If dizziness does occur, stop the treatment immediately.
- Talk to your doctor or therapist before using the product again.

The following is a description of a standard treatment session with the PARI O-PEP. Under certain circumstances, your doctor or therapist may recommend adopting a different treatment approach. In order to achieve the desired therapeutic outcome with the PARI O-PEP, always perform the treatment as instructed by your doctor or therapist.

- Hang the PARI O-PEP around your neck using the lanyard.
- Sit in an upright position and relax.
- Support the PARI O-PEP and hold it so that the mouthpiece is horizontal.
- Hold the mouthpiece between your teeth and enclose it with your lips.

i Make sure that the holes in the upper part are unobstructed and not covered by your hand.

- Breathe in as slowly and deeply as possible through your nose.
- Hold your breath for about 1 to 2 seconds.
- Breathe out slowly and completely through your mouth and into the PARI O-PEP (as slowly and for as long as possible). As you do so, tense your cheek muscles and keep your cheeks taut.
- Repeat the cycle of breathing in and out for 10 to 15 breaths. End the treatment session if the exertion becomes too great. The treatment will be ineffective if the patient breathes in too rapidly or has to exert himself excessively to breathe.

i Notes for optimum treatment:

Place your free hand flat against your chest while you breathe in and out. When you are using the device correctly, you will feel an unmistakable, regular vibration. The strength of the vibration varies according to the alignment of the PARI O-PEP. Find the alignment at which the vibration is strongest (tilt the PARI O-PEP slightly up or down).

4 REPROCESSING

If the product is used **in professional environments**, follow the information on reprocessing included in the appendix at the end of these instructions for use.

4.1 Reprocessing cycles

PARI O-PEP (incl. funnel and ball)	<ul style="list-style-type: none">- Clean immediately after every use- Disinfect at least once a day
Lanyard	Clean it whenever there is visible soiling [see: Care of the lanyard, page 20]. The lanyard cannot be disinfected.

4.2 Preparation

Dismantle the PARI O-PEP into its individual parts:

- Unscrew the upper part from the lower part.
- Take the ball out of the funnel. Make sure the ball does not roll away.
- Take the funnel out of the lower part.
- Detach the lanyard from the lower part.

4.3 Cleaning and disinfecting

Precleaning

All individual parts must be precleaned immediately after use.

EQUIPMENT:

- Drinking water temperature of about 15 °C

PROCEDURE:

- Rinse all parts used for 2 minutes in running drinking water beforehand.

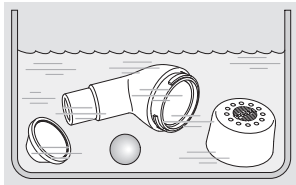
Cleaning

EQUIPMENT:

- Drinking water at a temperature from 40 °C to max. 50 °C
- Dishwashing liquid (e.g., Palmolive)
- Receptacle with at least 3 l capacity

PROCEDURE:

- Add about 1 teaspoonful to 3 l warm drinking water.
- Place all the parts in the rinsing water.
Treatment time: approx. 5 minutes

- Occasionally move the parts back and forth.
- In the case of heavy soiling, use a soft brush (e.g., a medium toothbrush which is used exclusively for this purpose).

RINSE:

- Rinse off all parts thoroughly in drinking water.
Water temperature: approx. 15 °C, duration: 2 minutes

DRY:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean and absorbent surface and allow them to dry completely.

Disinfecting

After cleaning, disinfect all of the **disassembled parts** (only parts that have been cleaned can be disinfected effectively).

The validated disinfection procedures are described below.

In boiling water

EQUIPMENT:

- Clean cooking pot
- Drinking water

PROCEDURE:

 **CAUTION**

Risk of infection due to moisture

Moisture encourages the growth of bacteria.

- Remove all parts from the pot as soon as the disinfection process is finished and allow them to dry.
- Place all the **individual parts** in boiling water (100 °C) for at least 5 minutes.

NOTE! Risk of damage to plastic parts! Plastic will melt if it comes into contact with the hot base of the pot. Make sure there is enough water in the pot to prevent the individual parts from touching the pot base.

DRY:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean and absorbent surface and allow them to dry completely.

Using a standard thermal disinfector for baby bottles (not a microwave oven)

EQUIPMENT:

- Thermal disinfector with a runtime of at least 6 minutes.

PROCEDURE:

 **CAUTION**

Risk of infection due to inadequate disinfection

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection.

- Make sure that the disinfector is clean and operating properly before every disinfection process.
- Allow the disinfection to continue until the disinfector switches off automatically or the minimum disinfection time stated in the instructions for use of the disinfector has elapsed. Do not switch the device off prematurely.

 **CAUTION**

Risk of infection due to moisture

Moisture encourages the growth of bacteria.

- Remove all parts from the disinfectant as soon as the disinfection process is finished and allow them to dry.

Regarding the performance of the disinfection, the duration of the disinfection procedure and the quantity of water required, follow the instructions for use of the disinfectant you are using.

DRY:

- After the disinfection process is complete, place all parts on a dry, clean and absorbent surface and allow them to dry completely.

Chemical disinfection

To ensure safety when handling chemicals, follow the instructions for use of the disinfecting agent, particularly the accompanying safety instructions.

EQUIPMENT:

- 1 sterilisation tablet
Active substance: Sodium dichlorisocyanurate
- 5 l drinking water (cold, no warmer than room temperature)
- Receptacle with at least 5 l capacity

Validated with Milton® sterilisation tablets

PROCEDURE:

 **CAUTION**

Risk of infection due to growth of bacteria

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection.

- Observe the mixing proportions indicated and the specified treatment time.
- Ensure that all components are completely submerged in the solution for the whole of the treatment time. There must not be any air pockets or bubbles.

- Dissolve one tablet in 5 l of cold drinking water.
- Place all the individual parts in the prepared solution.

Application time: 15 minutes

i *If the recommended application period is exceeded significantly, the plastic parts may take on the smell of the medium used.*

RINSE

CAUTION

Risk of allergic reactions and irritation of the mucous membrane by disinfectants

Disinfectants can trigger allergic reactions or irritation of the mucous membrane on contact with the skin.

- Rinse the product thoroughly to ensure that no residues of the disinfectant remain on the PARI product.
- Rinse off all parts thoroughly in running drinking water. Water temperature: approx. 15 °C, duration: 2 minutes
- Dispose of the used solution (the diluted solution can be disposed of down the drain).

DRY:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean and absorbent surface and allow them to dry completely.

4.4 Care of the lanyard

- Clean the lanyard as necessary with warm drinking water and a little dishwashing liquid.

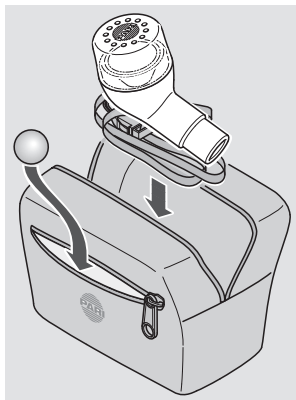
i *The lanyard cannot be disinfected.*

4.5 Inspecting

Inspect all product components after each cleaning and disinfection. Replace any broken, misshapen or seriously discoloured parts.

4.6 Storage

- Put the ball in the small side pocket.
- Reassemble the PARI O-PEP without the ball [see: Assembling the PARI O-PEP, page 13].
- Place the PARI O-PEP together with the lanyard in the case.

5 MISCELLANEOUS

5.1 Disposal

All product components can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective member countries.

5.2 Labelling

The following symbols can be found on the product and/or the packaging:

	Medical device
	Legal manufacturer
	Date of manufacture
	This product conforms to the EU Medical Device Regulation 2017/745.
	Consult instructions for use
	Item number
	Production batch number, lot

APPENDIX: Reprocessing in professional environments

Reprocessing cycles

Single patient use

PARI O-PEP (without lanyard)	<ul style="list-style-type: none">– Clean immediately after every use– Disinfect at least once a day
Lanyard	Clean it whenever there is visible soiling [see: Care of the lanyard, page 20]. The lanyard cannot be disinfected.

Before a change of patients

PARI O-PEP (without lanyard)	<ul style="list-style-type: none">– Cleaning– Disinfection– Sterilisation
Lanyard	Clean it whenever there is visible soiling [see: Care of the lanyard, page 20]. The lanyard cannot be disinfected.

Processing limits

300 processing cycles, max. 1 year

Separated parts for processing

All components of the PARI O-PEP can be cleaned, disinfected and sterilised according to the procedures described below. The lanyard must be processed separately [see: Care of the lanyard, page 20].

CAUTION

Risk of infection due to cross-contamination in the case of a change in patients

If a product is used for more than one patient, there is a risk that germs may be transmitted from one patient to the next.

- Clean, disinfect **and** sterilise all separated parts before every change in patient.

Preparation

Dismantle the inhalation device into its individual parts [see: Preparation, page 16].

Precleaning

All individual parts must be precleaned immediately after use.

EQUIPMENT:

- Drinking water temperature of about 15 °C

PROCEDURE:

- Rinse all parts used for 2 minutes in running drinking water beforehand.

Cleaning and disinfecting

Manual cleaning

EQUIPMENT

The method has been validated in Europe using:

- pH-neutral, enzymatic cleaner:
Bode Bomix® plus (concentration: 0.1%)
- Drinking water at a temperature from 15 °C to max. room temperature
- Application time: 10 minutes

PROCEDURE:

CAUTION

Risk of infection due to growth of bacteria

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection.

- Observe the mixing proportions indicated and the specified treatment time.
- Ensure that all components are completely submerged in the solution for the whole of the treatment time. There must not be any air pockets or bubbles.
- Clean the individual parts with a solution prepared according to the manufacturer's instructions.
For stubborn soiling, use a brush (e.g., a medium toothbrush which is used exclusively for this purpose).
- *If the recommended application period is exceeded significantly, the plastic parts may take on the smell of the medium used.*

RINSE:

- Rinse off all parts thoroughly in drinking water.
Water temperature: approx. 15 °C, duration: 2 minutes

DRY:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean and absorbent surface and allow them to dry completely.

Cleaning with disinfection

Cleaning and disinfection can be carried out in a single cycle using a chemical preparation process.

To ensure safety when handling chemicals, follow the instructions for use of the disinfecting agent, particularly the accompanying safety instructions.

Mechanical cleaning with disinfection:	<p>EQUIPMENT:</p> <p>The method has been validated in Europe using:</p> <ul style="list-style-type: none">– Alkaline cleaning agent: Dr. Weigert neodisher® MediClean forte (concentration: 0.5%)– Deionised water– Cleaning and disinfection device: Steelco DS800 (in conformance with DIN EN ISO 15883-1 and 15883-2) <p><i>Info:</i> <i>If a different alkaline cleaning agent is used, it may also be necessary to use a neutralising agent. If applicable, follow the recommendations of the manufacturer of the chemical.</i></p> <p>PROCEDURE:</p> <p>Programme for cleaning and disinfecting according to manufacturer's instructions.</p> <p>DRY:</p> <p>Ensure that there is no residual moisture remaining in the components. Optionally:</p> <ul style="list-style-type: none">• Shake the water out of all of the parts.• Place all parts on a dry, clean and absorbent surface and allow them to dry completely.
---	---

**Chemical
cleaning with
disinfection:**

EQUIPMENT:

The method has been validated in Europe using:

- Aldehyde-free instrument disinfectant: Bode Bomix® plus (concentration: 2%)
Active agent basis: Quaternary ammonium compound
- Cold drinking water (no warmer than room temperature)
- Application time: 5 minutes

PROCEDURE:

- Clean and disinfect the individual parts in a single work step with a solution prepared according to the manufacturer's instructions.
***Info:** If the recommended treatment time is exceeded significantly, the plastic parts may take on the smell of the disinfectant.*

RINSE:

- Rinse off all parts thoroughly in running drinking water.

Water temperature: approx. 15 °C, duration: 2 minutes

 **CAUTION!** Residues of the disinfectant can cause allergic reactions or irritation of the mucous membrane.

- Dispose of the used solution (the diluted solution can be disposed of down the drain).

DRY:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean and absorbent surface and allow them to dry completely.

Chemical disinfection

To ensure safety when handling chemicals, follow the instructions for use of the disinfecting agent, particularly the accompanying safety instructions.

EQUIPMENT:

The method has been validated in Europe using:

- Aldehyde-containing disinfectant: Bode Korsolex® basic (concentration: 4%)
Active agent basis: Aldehyde donor, aldehyde¹
- Drinking water (cold, no warmer than room temperature)
- Application time: 30 minutes

PROCEDURE:

CAUTION

Risk of infection due to growth of bacteria

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection.

- Observe the mixing proportions indicated and the specified treatment time.
 - Ensure that all components are completely submerged in the solution for the whole of the treatment time. There must not be any air pockets or bubbles.
 - Disinfect the individual parts with a solution prepared according to the manufacturer's instructions.
- i** *If the recommended application period is exceeded significantly, the plastic parts may take on the smell of the medium used.*

1) Additional activity spectra: tuberculocidal, mycobactericidal, fungicidal
According to the information supplied by the disinfectant manufacturers, the agents listed are effective against the gram-positive bacteria *Staphylococcus aureus* and *Enterococcus hirae* and against the gram-negative bacteria *Escherichia coli*, *Pseudomonas aeruginosa* and *Proteus mirabilis* as well as the yeast-like fungus *Candida albicans*.

RINSE

CAUTION

Risk of allergic reactions and irritation of the mucous membrane by disinfectants

Disinfectants can trigger allergic reactions or irritation of the mucous membrane on contact with the skin.

- Rinse the product thoroughly to ensure that no residues of the disinfectant remain on the PARI product.
- Rinse off all parts thoroughly in running drinking water. Water temperature: approx. 15 °C, duration: 2 minutes
- Dispose of the used solution (the diluted solution can be disposed of down the drain).

DRY:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean and absorbent surface and allow them to dry completely.

Sterilising

CAUTION

Risk of infection by remaining germs

If any germs are left on the parts that are to be sterilised, they can proliferate despite sterilisation. As a result, there is a danger of infection.

- Clean, disinfect and dry all components thoroughly before sterilising.
- Use only validated procedures for cleaning and disinfection.

EQUIPMENT:

The method has been validated in Europe using:

- Steam steriliser with fractionated pre-vacuum (in accordance with DIN EN 285 or DIN EN 13060 (Type B))
- Sterile barrier system in accordance with DIN EN 11607-1

- Temperature: 132 °C / 134 °C
- Holding time: 3 minutes

PROCEDURE:

- Pack all of the disassembled parts in a sterile barrier system in accordance with DIN EN 11607-1 (e.g., foil-paper packaging).
- Carry out the sterilisation in a steam steriliser in accordance with the manufacturer's instructions.

Sterilisation temperature and holding time:
132 °C / 134 °C, at least 3 minutes.

DRY:

Ensure that there is no residual moisture remaining in the components. Optionally:

- Shake the water out of all of the parts.
- Place all parts on a dry, clean and absorbent surface and allow them to dry completely.

Visual inspection and storage

Storage location:

- dry
 - dust-free
 - protected from sources of contamination
- optional: Use sterile packaging

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