easy care

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en Instructions for Use

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da Brugsanvisning

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
easy care Cleaning aid for aerosol heads

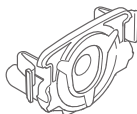Catalogue number 078G6100

1. Important information

Read these instructions for use **carefully and completely** before using for the first time. Keep these instructions for use safely so that you can refer to them again later. If these instructions for use are not followed, damage to the product cannot be ruled out.

Report serious incidents to the manufacturer and the competent regional authorities.

Clean the *easy care* as described in the section entitled Cleaning before using for the first time.

The connection cord and the controller are also required for operation. Notes on preparing the controller can be found in the instructions for use of the eFlow[®]*rapid* nebuliser system.

Cleaning with the *easy care* cleaning aid is recommended if the nebulisation time for the same fill volume of a given medication increases significantly (doubles). Additional information can be found in the current version of the eFlow[®]*rapid* instructions for use in the chapter “TROUBLESHOOTING”.

Use the cleaning aid as necessary, up to twice a week after an inhalation session and after cleaning the aerosol head (see sequence of steps in a backwash procedure). More frequent use is not beneficial, but can instead lead to premature material wear.

During its entire life cycle, an aerosol head should not be backwashed more than 25 times with the cleaning aid.

The cleaning aid does not extend the life cycle of the aerosol head.

2. Product description

Intended purpose

The *easycare* cleaning aid is intended to clean the membrane of the aerosol head of eFlow® Technology nebuliser handsets by backwashing.

Intended use

The cleaning aid must only be used in accordance with its intended purpose. No areas of use have been defined for the cleaning aid, and no contraindications to its use are known. Backwashing unblocks the pores in the membrane of the aerosol head by flushing with liquid in the opposite direction to the direction for nebulising.

For a backwashing procedure, you need 5 ml backwash liquid. Only use one of the liquids listed below as the backwash liquid:

- 5 ml isotonic sodium chloride solution (0.9%)
- 5 ml distilled water
- 5 ml deionized water

Info

Backwashing is no substitute for the steps described in the eFlow®rapid instructions for use regarding cleaning and disinfection.

The steps of the backwash procedure must be carried out in the following order without fail:

1. Cleaning the nebuliser handset parts and the aerosol head*)
2. Backwashing the aerosol head with *easycare*
3. Disinfecting the nebuliser handset parts and the aerosol head*)

*) Regarding items 1 and 3 see Chapter "CLEANING AND DISINFECTION" in the instructions for use of the eFlow[®]rapid nebuliser system.

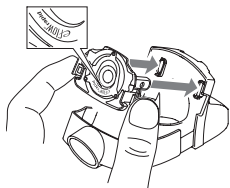
Life cycle

The expected life cycle of the *easycare* cleaning aid is 3 years.


3. Preparing for the backwash procedure


- Open the cleaning aid.
To do this, press lightly on the lateral gripping surfaces.
- Unfold the parts.

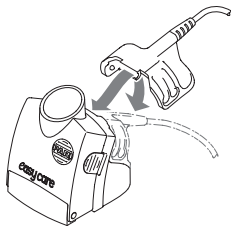- Insert the aerosol head.
The aerosol head must be felt to snap into place upon insertion. Make sure it is positioned in the right direction.
- Close the cleaning aid. When closing, ensure that the two lateral engaging hooks on the cleaning aid are felt to snap into place.

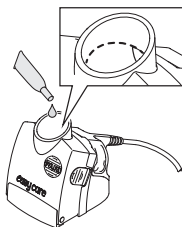- Connect the cleaning aid to the controller.
To do this, attach the nebuliser connector of the connection cord to the hinge of the cleaning aid and swivel it downwards so that the metal contacts engage. Connect the other end of the cord to the controller.

- Place the cleaning aid on a level surface.
- Pour backwash liquid into the funnel opening until it reaches the marking (about 5 ml).

4. Performing the backwash procedure

- Start the backwash procedure by pressing the ON/OFF button on the controller.

The processing time for 5 ml is at least 10 minutes.

After backwashing, the controller switches off automatically. If there is still some liquid in the funnel, start the backwash procedure again.

i Info

Aerosol may still escape from the funnel opening of the cleaning aid at the end of the backwash procedure. This is normal. The controller can then also be switched off manually by pressing the ON/OFF button.

5. Cleaning

- Disconnect the connection cord from the cleaning aid and the controller.
- Open the cleaning aid and remove the aerosol head by pressing on the two lateral engaging hooks (for disinfection of the aerosol head, see Chapter "CLEANING AND DISINFECTION" in the instructions for use of the eFlow[®]rapid nebuliser system).
- Clean the cleaning aid with warm tap water and dishwashing soap after each use.
- Then rinse the cleaning aid in warm tap water.
- Place the cleaning aid open and upside down on a clean surface to dry completely in the air.

6. Explanation of symbols

	Medical device
	Please follow the instructions for use
	Item number
	Lot number
	Single Patient – Multiple use
	CE marking: This product is compliant with the requirements set forth in Regulation (EU) 2017/745 on medical devices.
	Caution

	Date of manufacture
	Manufacturer

We reserve the right to make technical changes.

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CE-işareti: Bu ürün, 2017/745 sayılı AB Tıbbi Cihaz Yönetmeliği kapsamındaki gereklilikleri karşılar özelliindedir.
PARI Pharma GmbH