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en Instructions for use eFlow[®]rapid nebuliser handset

i – Read these instructions carefully to learn how to use your eFlow[®]rapid nebuliser handset.
– Make sure you also read the **instructions for use enclosed with the eFlow[®]rapid nebuliser system before first use.** They include important additional information about use of the eFlow[®]rapid nebuliser handset in particular in relation to connection, operation and hygiene.

1 Intended use

The eFlow[®]rapid nebuliser handset is intended to be used in combination with an eFlow[®]rapid control unit (type 178) or an eBase Controller (type 678).

! The eFlow[®]rapid nebuliser handset is intended for single patient use at home.

The nebulisation time depends on the inhalation product and fill volume.

2 Use of nebuliser handset

- Disassemble and clean the nebuliser handset including the aerosol head according to the instructions for use enclosed with the eFlow[®]rapid nebuliser system before first use.
- Assemble the nebuliser handset as shown in the pictures:
 - Open the nebuliser handset
 - Insert the aerosol head – **Do not touch the membrane (metal part, CENTER) of the aerosol head!**
 - Place the inspiratory valve
 - Close the nebuliser handset
 - Attach the mouthpiece
 - Mount the connection cord
 - Fill in the medication
 - Close the medication reservoir
 - Sit in an upright position and relax
This makes inhalation easier and improves deposition of the medication in the respiratory tract.

3 Cleaning and disinfection

The nebuliser handset (incl. aerosol head) must be cleaned immediately after each use and disinfected at least once a day (see instructions for use enclosed with the eFlow[®]rapid nebuliser system).

4 Replace your nebuliser handset

The nebuliser is designed for multiple uses. In order to ensure that an exact quantity is delivered throughout the inhalation treatment period, and that the session is as quick as possible, the nebuliser must be replaced at least every 12 months and the aerosol head every 3 to 6 months depending on the frequency and duration of its use.

5 Package contents

One eFlow[®]rapid nebuliser handset including an eFlow[®]rapid aerosol head.

6 Explanation of symbols

	Manufacturer
REF	Article number
SN	Serial number
CE 0123	CE marking: This product is compliant with the requirements set forth in 93/42/EEC (Medical Devices) and 2011/65/EU (RoHS).
	Safety warning
	Follow the instructions for use (background = blue, symbol = white)
	The medical device was distributed commercially after 13 August 2005. This product must not be disposed of with normal domestic waste. The symbol of a crossed-through refuse bin indicates that separate collection is required.

7 Particle size distribution

Information about particle size distribution compliant with DIN EN 13544-1 is available on request from PARI Pharma.

Further information is available from your local service partner.
www.eflowrapid.info

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