
REF 678G8210

CE 0123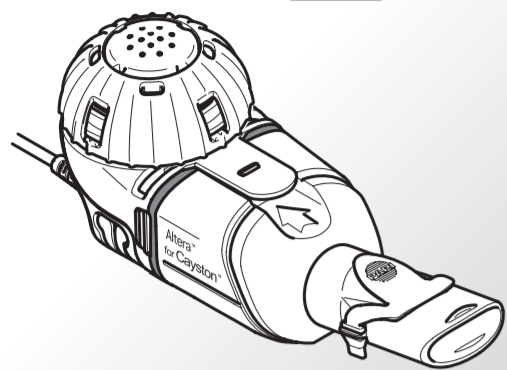

<b>de</b>	Gebrauchsanweisung	<b>fr</b>	Notice d'utilisation
<b>en</b>	Instructions for Use	<b>it</b>	Istruzioni per l'uso
<b>da</b>	Brugsanvisning	<b>nl</b>	Gebruiksaanwijzing
<b>es</b>	Instrucciones de uso	<b>sv</b>	Bruksanvisning

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## en Instructions for use Altera® nebuliser handset

- i** – Read these instructions for use carefully and completely.  
– Read the **instructions for use of the eFlow®rapid nebuliser system completely before using for the first time. These contain further important instructions for the use of the Altera® nebuliser handset, particularly regarding connection, operation, cleaning and disinfection.**

### 1 Intended purpose

The Altera® nebuliser handset with an eBase® Controller is intended to administer Cayston® as an aerosol for inhalation.

### 2 Indication

The Altera® nebuliser handset is indicated to administer Cayston® to treat lung infections caused by *Pseudomonas aeruginosa* in patients with cystic fibrosis (CF).

### 3 Contraindication

For the Altera® nebuliser handset no contraindication is known to PARI Pharma GmbH. Take note of contraindications of the medicine in the Cayston® package leaflet.

### 4 Use of the nebuliser handset

The Altera® nebuliser handset must only be used in accordance with its intended purpose. For reasons of hygiene and to avoid cross-infections, the Altera® nebuliser handset must only be used by one patient.

- Before using for the first time, disassemble and clean the nebuliser handset including the aerosol head as described in the instructions for use of the eFlow®rapid nebuliser system.
- Assemble the nebuliser handset as shown in the diagram:
  - Open the nebuliser handset.
  - Insert the aerosol head. **Do not touch the membrane (the curved metal surface in the middle of the aerosol head).**
  - Insert the inspiratory valve.
  - Close the nebuliser handset.
  - Attach the mouthpiece.
  - Connect the connection cord.
  - Fill with the medication.
  - Close the medication reservoir.
  - Sit in an upright position and relax. This makes inhaling easier and improves uptake of the medication in the airways.

**!** Hold the nebuliser handset horizontally all the time while performing the inhalation. If the handset is tilted, errors in the dosage may occur.

- Hold the mouthpiece between your teeth and enclose it with your lips. The lips must not touch the blue expiratory valve.

The inhalation session should last between 2 and 3 minutes. Inhalation is ended as soon as no more aerosol is generated.

- Always check the medication reservoir to ensure that the entire dose of Cayston® has been nebulised.

### 5 Cleaning and disinfection

The dismantled nebuliser handset (including the aerosol head) must be cleaned immediately after each use and disinfected at least once a day (additional information can be found in the instructions for use of the eFlow®rapid nebuliser system).

### 6 Replacing the nebuliser handset

Each 28-day pack of Cayston® contains an Altera® nebuliser handset. Use the new nebuliser handset for each treatment cycle.

### 7 Package contents

One Altera® nebuliser handset with one Altera® aerosol head. There are no accessories for the Altera® nebuliser handset.

### 8 Symbols

<b>MD</b>	Medical device
	Consult instructions for use
	Single patient – Multiple use
<b>LOT</b>	Batch code
<b>REF</b>	Item number
<b>SN</b>	Serial number
	Minimum and maximum ambient temperature
	Minimum and maximum atmospheric humidity
	Minimum and maximum atmospheric pressure
CE 0123	CE marking: This product is compliant with the requirements set forth in Directive 93/42/EEC (Medical Devices).
	The medical device was distributed commercially after 13 August 2005. <b>This product must not be disposed of with normal domestic waste.</b> The symbol of a crossed-through refuse bin indicates that separate collection is required.
	Caution
	Manufacturer
	Date of manufacture

### 9 Service and contact

In case of technical problems or questions about the device, please contact your service partner. The contact addresses of your responsible service partners as well as a digital version of your Altera® instructions for use can be found at [www.pari.com/altera](http://www.pari.com/altera).

Aerosol data is available from PARI Pharma upon request.

We reserve the right to make technical changes.  
Information as of: 2020-12

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